

Zenit PRO – Maintenance Form

Site: _____

Instrument S/N: _____

Instructions: Ensure each of the following maintenance items is performed either Daily, Weekly or Monthly, as indicated. Additionally document each “As needed” maintenance item performed. Initial each item as it is completed under the appropriate date. If any item is not applicable, Enter N/A.

Daily Maintenance	Date Performed:				
Check/refill system 5L tank					
Check Liquid waste tank					
Check Safety Drain container					
Check tubes connected to the fluidic ports					
Check volumetric syringe tube for bubbles, leaks or visible damage					
Confirm Sample, Control, Diluent, Reagent, Deepwell plate and slide parking racks are clear					
Perform end of day wash (1 cycle)					

Weekly Maintenance	Date Performed:				
Disinfect external surfaces with 70% ethanol using lint free wipes					
Disinfect computer touch screen with 70% isopropyl ethanol using lint free wipes					
Perform end of day wash (3 cycles)					

Montly Maintenance	Date Performed:				
Perform end of day wash with 0.1% bleach					
Clean the tanks with tap water					
Clean the tanks with DI water					
Clean the waste tank					

As needed Maintenance	Date Performed:				
Comments:					

Reviewed by: _____

Date: _____