

PROJECT ID	PR_M_20140318_1	PRODUCT	Zenit PRO
REVISION N.	1	DATE	03/07/2023
SW version	1.6.21	Note	
	NAME	ROLE	SIGNATURE
PREPARED BY	M. MEONI	R&D Manager and Project Leading	
VERIFIED BY	G. DONATI	Technical Office, Design & Engineering Manager	
APPROVED BY	S. FANTECHI	Operation & QA/RA Manager	

REVISION HISTORY

Revision Number	Revision Date	Revision Note
1	03/07/2023	First emission

Zenit PRO – Software version 1.6.21

1. Major defects corrected and major features introduced in software version 1.6.21 (since last release 1.6.20)

Defect/Feature	Description
VR-639	Bug fixed on gripper hw1 timeout
VR-640	HL7 service startup procedure modified in order to be immediately ready for inbound messages
VR-641	Bug fixed in archive search button with UI language different from English or Italian
VR-647	Editable positive and negative thresholds for ANCA analysis added in configuration module
VR-647	ANCA module cells computation timing optimized
VR-686	Bug fixed on LIS xml requests with new patients

2. Known unresolved anomalies in software version 1.6.21

Defect	Description	Workaround / Preventative Measure